



# WalkMed








## 350VL Ambulatory Pumps

### Patient Quick Reference Card

Keep this card in your carrying pouch or on your person. For complete guidance, refer to the instructions given to you by your health care provider.

#### Brief Instructions

##### To Start Medication Delivery:



1. Press the POWER  key to turn pump on. The pump will display .
2. Press the START  key to begin delivery. The   light will start blinking green. The display will alternate between delivery rate  and volume delivered  screens.
3. Place front cover on pump.

##### To Stop Medication Delivery:

1. Press the STOP  key. The pump will display .

##### To Change the Battery:

Use a 9-volt alkaline battery. Duracell® MN 1604 recommended.

1. Press the STOP  key and turn pump off .
2. Remove battery. Wait approximately 15 seconds before installing new battery.
3. Follow **Start Medication Delivery** instructions above.

##### Additional Information:

---

---

---

---

##### Authorized Representative:

Priory Analysts Ltd.  
The Atrium, Park St. West  
Luton, LU1 3BE, United Kingdom

##### Manufactured by:

#### WalkMed Infusion LLC

96 Inverness Drive East, Unit J  
Englewood, Colorado 80112

Phone: (303) 420-9569 Fax: (303) 420-4545  
C€1275



# Problem Solving Table

Software Revision Level 2.0 [.1-.9]

1. Match the message you see on your pump display with the **Display Message** in the left column below.
2. Follow the **Corrective Action** given in the right column.
3. If you cannot correct the problem following the **Corrective Action**, follow the directions given by your health care provider.

( \_\_\_\_\_ ) \_\_\_\_\_ - \_\_\_\_\_

Display Message	Sounds/ Lights	Condition	Corrective Action
<p>idle</p> <p>Or</p> <p>888</p>	<p>Sounds: Slow beeping</p> <p>Light:</p> <p>Blinking yellow</p>	Not delivering	<p>Press  key to start delivery.</p> <p>Press  key to silence alert for 5 minutes.</p>
	<p>Sounds: Slow beeping</p> <p>Light:</p> <p>Blinking yellow</p>	Low battery	<p>Press  key to silence alert.</p> <p>Replace battery within 30 minutes.</p>
<p>dep</p>	<p>Sounds: Continuous tone</p> <p>Light:</p> <p>Continuous red</p>	Depleted battery	<p>Remove depleted battery to silence alarm.</p> <p>Replace battery.</p> <p>Press  key to turn pump on.</p> <p>Press  key to restart.</p>
<p>dop</p>	<p>Sounds: Continuous tone</p> <p>Light:</p> <p>Blinking red</p>	Clamp bar is open	<p>Press  key to silence alert.</p> <p>Close clamp bar.</p> <p>Press  key to restart.</p>
<p>occl</p>	<p>Sounds: Continuous tone</p> <p>Light:</p> <p>Blinking red</p>	I.V. tubing occlusion	<p><b>Automatic:</b> Check tubing system for kinks and correct.</p> <p>Pump will resume delivery when line is cleared.</p> <p><b>Manual:</b> Press  key to silence alert.</p> <p>Check tubing system for kinks and correct.</p> <p>Press  key to restart.</p>
<p>Err.</p>	<p>Sounds: Continuous tone</p> <p>Light:</p> <p>Continuous red</p>	System malfunction	<p>Write down Error message from display.</p> <p>Remove battery to silence alarm.</p> <p>Take pump out of service.</p> <p>Contact support with Error message.</p>
<p>scrolling</p> <p>End</p> <p>0.50</p> <p>10.5</p>	<p>Sounds: Slow beeping</p> <p>Light:</p> <p>Blinking yellow</p>	Total Volume Limit reached	<p>The pump will continue delivery at the KVO rate for approximately 4 hours.</p> <p>An audible alert will beep every 30 minutes.</p>
<p>End</p>	<p>Sounds: Fast beeping</p> <p>Light:</p> <p>Blinking red</p>	End of KVO	<p>Press  key to silence alert.</p> <p>Press the  key to turn the pump off.</p>