



WalkMed® TRITON™ Pole Mount Infusion Pump

Durable, Compact, Easy to Use

The TRITON infusion pump is a durable, compact, pole mount infusion pump focused on ease of use. The simple programming structure and large display screen make the pump an attractive choice for care providers in a wide range of patient care settings.

The TRITON is designed for infusion of medications or fluids requiring continuous delivery through clinically acceptable routes of administration: intravenous (central and peripheral), subcutaneous and intra-arterial.

Operational Highlights

- Large, Easy to Read Display and 360° Viewable Indicator Lights
- Extended Battery Life – “Full Day” Dependability
- Quiet Pumping Mechanism and +/- 5% Accuracy

Safety & Convenience

- Event Log
- Built-in & Adjustable Air, Upstream & Downstream Pressure Sensors
- Customizable Pump Settings for Individual Patients & Protocols

Need help with a WalkMed brand product? Call our 24-Hour Support Line: (866) 484-5601.

CAUTION: Federal (USA) law restricts this device to sale by, or on the order of a physician.



Indicator Lights
(Red in Stop or Alarm State; Green during infusion or programming state)

Large, Easy to Read, Backlit Display Screen

Four Direction Scrolling

Prime Button

ON/OFF Button

Power Indicator Light

A & B Selection Buttons

Home Screen Button

Programming


Continuous Mode – Primary Infusion
Specific volume over a programmed time (vol/time) or specific rate (mL/hr).

Continuous Mode – Secondary Infusion
Specific rate (mL/hr) over a programmed volume (mL). Pump will automatically resume the primary infusion when the secondary infusion is completed.

Ramping Mode
Programmable steps to titrate rate (mL/hr).

Specifications

Pumping Mechanism	Linear Peristaltic
Flow Rate	1-1200 mL/hr in 1 mL increments
Priming Rate	500 mL/hr
Volume	1-9999 mL
Accuracy	+/- 5%
Air Detector	0.1 - 2.0 mL
Up/Down Pressure Sensors	Adjustable Sensitivity
Infusion Modes	Continuous Mode, Ramping Mode, Primary and Secondary Infusions
Weight	6.6 lbs
Size	9.5 x 4.7 x 7.7 in (h/w/d)

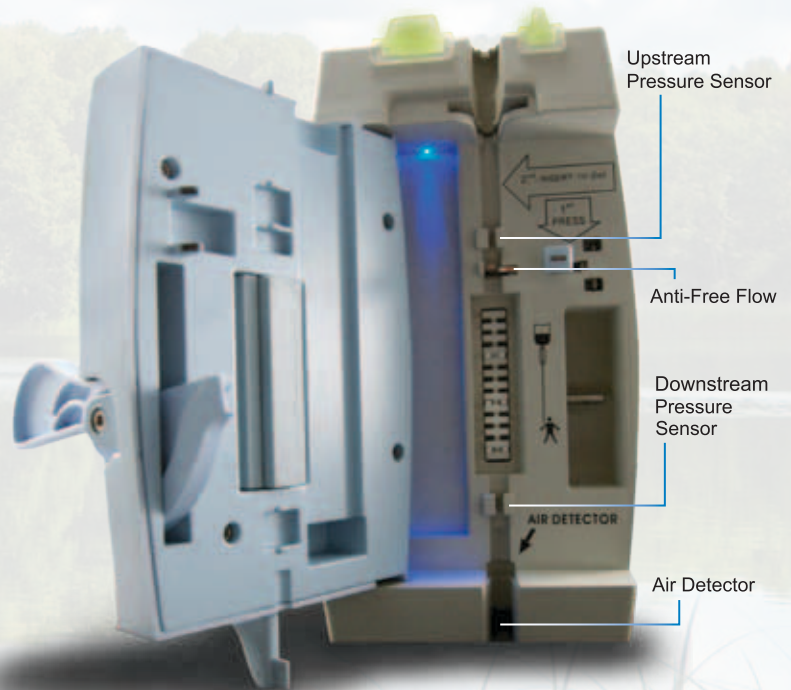

Alarms: Up/Down Pressure Sensors; Near End; Infusion Complete; KVO; Low Flow; Battery Low; No Power; Air In Line; Door Open; System Error; Alarm Time Counter; Hold Mode

WalkMed Infusion

96 Inverness Drive East, Englewood, Colorado 80112

Phone: (303) 420-9569 Fax: (303) 420-4545

walkmed.net